[a]: 清洁剂	[b]: 擦拭类			[c]: 喷洒类		[d]: 溶液类			[e]: 设备类	[f]: 粉末类
a1: MetriZyme a2: Tristel Pre-Clean Wipes a3: Liquinox a4: Revital-OX Enzymatic Detergent a5: MetriSponge a6: Prolystica 2X Concentrate Enzymatic Cleaner a7: Endozime and Endozime Sponge a8: klenzyme a9: ANIOSYME 5 a10: DDN9	b1: CLEANISEPT® WIPES b2: mikrozyd® AF Wipes Jumbo b3: PROTEX™ b4: DISINFECTANT Wipes b5: Sani-Cloth® Plus SONO™ b7: ULTRASOUND WIPES b8: Tristel Sporicidal Wipes b9: Tristel Rinse Wipes b10: Clinell® Universal Wipes/Clinell® Surface Wipes b11: mikrozyd® Sensitive Wipes b12: Wip'Anios premium b13: Sani-Cloth AF3 (gray) b14: Protex Ultra Wipes b15: Sani-Cloth HB	b16: CaviWipes b17: Dispatch Towels b18: Accel TB Wipes b19: CaviWipes 1 b20: Tuffle 5 b21: Sani-Cloth Active b22: Septiwipes b23: Mikorbac Tissues b24: Sani-Cloth Germicidal Wipes b25: WIP'ANIOS CLEAN'UP b26: CaviWipes XL b27: OXIVIR™ WIPES b28: Oxivir 1 Wipes b29: PDI SANI-CLOTH BLEACH WIPES b30: wip anios excel b31: Virusolve® + Pronte all'uso Wipes	b32: Super Sani-Cloth® b33: SANI-CLOTH® PRIME WIPES	c1: Oxivir™/MC Tb c2: PI-SPRAY II c3: Surfa'safe c4: TRANSEPTIC c5: PROTEX™ c7: Tristel Duo c8: IODOCLEAN c9: Protex Spray c10: CaviCide c11: T-Spray c12: Indican Form c13: Transeptic Spray c14: CAVICIDE 1 c15: OXIVIR 1 c16: WIP'ANIOS SPOR'ACTIV c17: Accel TB Liquid	c18: Sani-HyPerCide GERMICIDAL SPRAY c19: Sani-24 GERMICIDAL SPRAY	d1: CIDEX OPA d2: Cidex Activated Dialdehyde Solution d3: Minncare® Cold Sterilant d4: Ster-Bac d5: Triacid-N d6: Revital-Ox® Resert® High Level Disinfectant d7: Gigasept® PAA concentrate d8: DESCOTON extra d9: Gigasept® FF(neu) d10: ANIOXYDE 1000 d11: SALVANIOS pH10	d12: Cavicide Liquid d13: Metricide d14: Metricide 28 d15: Metricide OPA Plus d16: Cidex Plus d17: Gigasept AF d18: Osvan d19: Neojodin d20: Milton d21: ibitane d22: Sterihyde d23: Metricide 14 d24: Sekusept plus d25: Wavicide-01 d26: SALVANIOS pH7	d27: Minncare liquid disinfectant d28: Virusolve® + Concentrate d29: Virex II 256 High-Level disinfectant d30: UltrOx™	e1: TrophonSonex-HL (Used with Trophon/Trophon2) e2: Germitec UV-C e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System) e4: STERRAD® system	f1: Rely+On PeraSafe

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[a]: 清洁剂	[b]: 擦拭类			[c]: 喷洒类			[d]: 溶液类			[e]: 设备类	[f]: 粉末类
a1: MetriZyme a2: Tristel Pre-Clean Wipes a3: Liquinox a4: Revital-OX Enzymatic Detergent a5: MetriSponge a6: Prolystica 2X Concentrate Enzymatic Cleaner a7: Endozime and Endozime Sponge a8: klenzyme a9: ANIOSYME 5 a10: DDN9	b1: CLEANISEPT® WIPES b2: mikrozid® AF Wipes Jumbo b3: PROTEX™ DISINFECTANT Wipes b4: Sani-Cloth® Plus b5: SONO™ ULTRASOUND WIPES b7: Tristel Sporicidal Wipes b8: Tristel Rinse Wipes b9: Clinell Universal Wipes/ Clinell® Surface Wipes b10: mikrozid® Sensitive Wipes b11: Wip'Anios premium b12: ultrasound probe cleaning wipes b13: Sani-Cloth AF3 (gray) b14: Protex Ultra Wipes b15: Sani-Cloth HB	b16: CaviWipes b17: Dispatch Towels b18: Accel TB Wipes b19: CaviWipes 1 b20: Tuffle 5 b21: Sani-Cloth Active b22: Septiwipes b23: Mikorbac Tissues b24: Sani-Cloth Germicidal Wipes b25: WIP'ANIOS CLEAN'UP b26: CaviWipes XL b27: OXIVIR™ WIPES b28: Oxivir 1 Wipes b29: PDI SANI-CLOTH BLEACH WIPES b30: wip anios excel b31: Virusolve® + Pronte all'uso Wipes	b32: Super Sani-Cloth® b33: SANI-CLOTH® PRIME WIPES	c1: Oxivir™/MC Tb c2: PI-SPRAY II c3: Surfa'safe c4: TRANSEPTIC c5: PROTEX™ DISINFECTANT SPRAY c7: Tristel Duo c8: IODOCLEAN c9: Protex Spray c10: CaviCide c11: T-Spray c12: Indican Form c13: Transeptic Spray c14: CAVICIDE 1 c15: OXIVIR 1 c16: WIP'ANIOS SPOR'ACTIV c17: Accel TB Liquid	c18: Sani-HyPerCide GERMICIDAL SPRAY c19: Sani-24 GERMICIDAL SPRAY	d1: CIDEX OPA d2: Cidex Activated Dialdehyde Solution d3: Minncare® Cold Sterilant d4: Ster-Bac d5: Triacid-N d6: Revital-Ox® Resert® High Level Disinfectant d7: Gigasept® PAA concentrate d8: DESCOTON extra d9: Gigasept® FF(neu) d10: ANIOXYDE 1000 d11: SALVANIOS pH10	d12: Cavicide Liquid d13: Metricide d14: Metricide 28 d15: Metricide OPA Plus d16: Cidex Plus d17: Gigasept AF d18: Osvan d19: Neojodin d20: Milton d21: ibitane d22: Sterihyde d23: Metricide 14 d24: Sekusept plus d25: Wavicide-01 d26: SALVANIOS pH7	d27: Minncare liquid disinfectant d28: Virusolve® + Concentrate d29: Virex II 256 High-Level disinfectant d30: UltrOx™	e1: TrophonSonex-HL (Used with Trophon/ Trophon2) e2: Germitec UV-C e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System) e4: STERRAD® system	f1: Rely+On PeraSafe	

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