

Importing Configurations Mindray ePM Patient Monitor

How to download ePM Standard configurations from the Mindray UK website:


- Visit www.mindrayuk.com/education
- Find the ePM Standard Configuration zip file and download onto a PC. Please note an encrypted USB stick cannot be used
- Unzip the file and copy the **mrbigdipercfg** folder and the **mrbigdiperUser** folder to a USB stick

To import the configuration from the USB drive to the monitor, follow this procedure:

- Turn the ePM patient monitor on with power key
- Connect the USB drive to the monitor's USB port on the back of the monitor
- Select the **Main Menu** quick key




- Swipe with 2 fingers to the **Configuration** column → select **Manage**

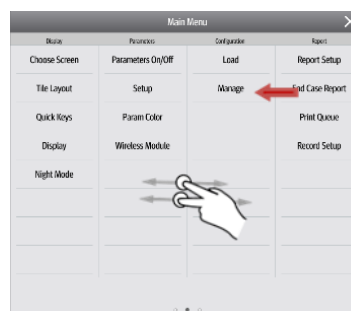



- Input the password **315666** and select

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.



- Ensure the monitor shows **General**, Default Patient Category is **Adult** and Default Use EWS is **Yes**
See image below:

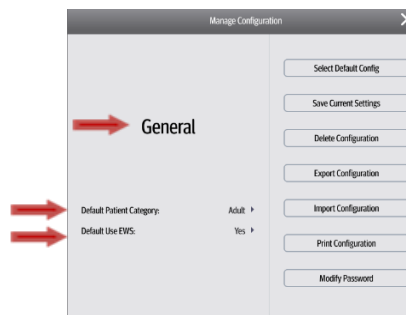

- Select **Import Configuration** → select **User Maintenance Settings, ePM Standard (Adult), ePM Standard (Ped)** and **ePM Standard (Neo)** → select **Import** → select **Back**

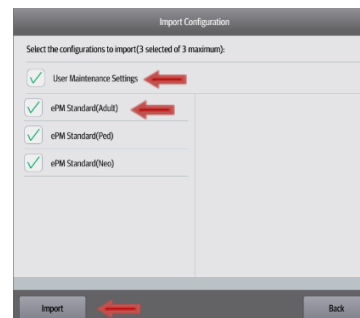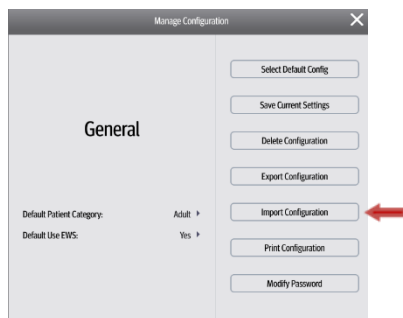

- In the **Manage Configuration** menu choose **Select Default Config** → select **Load Specified Config**

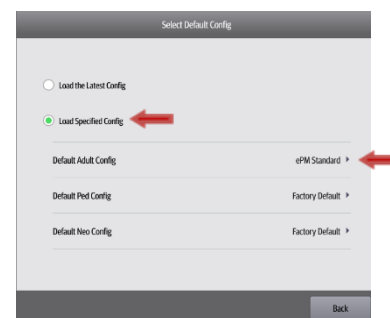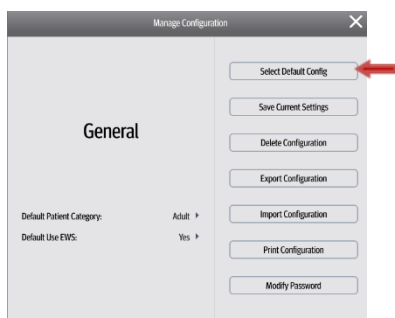

- Press the area labelled **Factory Default** located next to **Default Adult Config / Default Ped Config / Default Neo Config** → select **ePM Standard** for each of the three patient categories (Adult, Ped and Neo)
- Press **'X'** to exit menu

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.



- Select **Alarm Setup** key



- Select the **Setup** tab



- Select switch next to **Auto Limits for New Patient** and turn off
See image below:

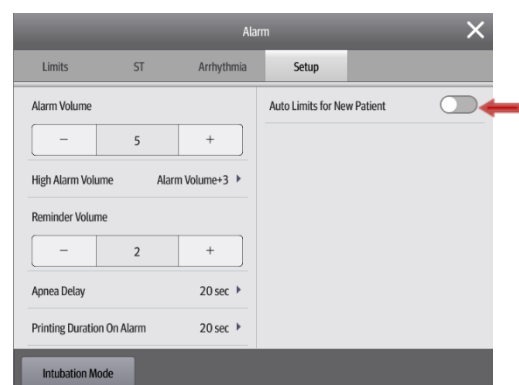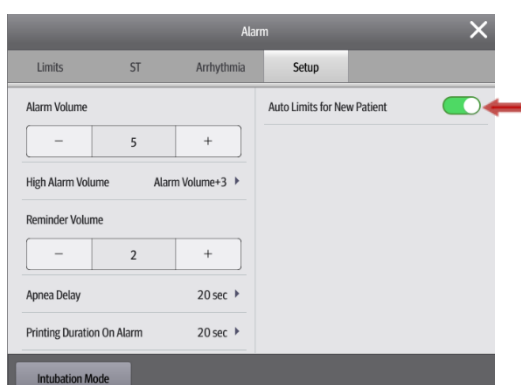



- Turn the device off



These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.

To export the configuration to the USB drive from the monitor, follow this procedure:

- Connect the USB drive to the monitor's USB port.
- Select the **Main Menu** quick key → swipe to the **Configuration** column → select **Manage**.

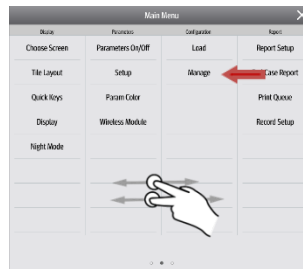

- Input the password **315666** and select 
- Select **Export Configuration** → select the **Configurations** and **User Maintenance Settings** to Export → select **Export** → select **Back**.

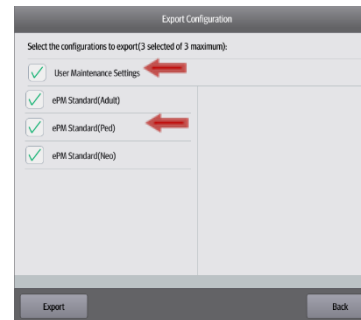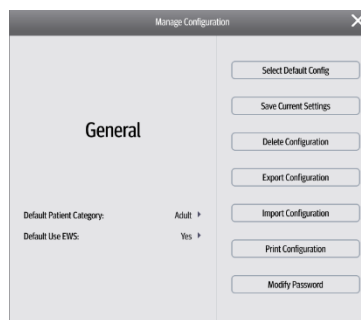

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.