


BeneVision N1 Monitor

Quick Guide

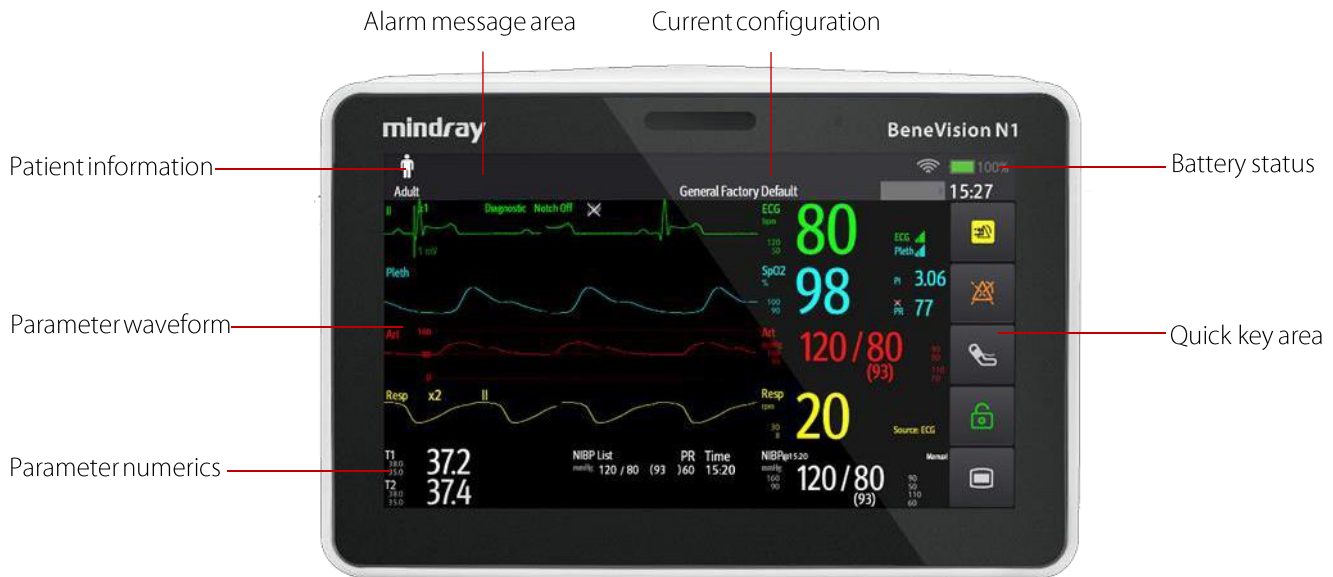



Removing the N1 from the host monitor, dock or module rack

- Press and hold the latch at the bottom of the N1
- Slide the N1 from the host monitor, dock or module rack

Inserting the N1 into the host monitor, dock or module rack

- Slide the N1 into the docking area located to the side of the host monitor, dock or module rack
- An audible click indicates that the N1 is docked

To lock or unlock the touch screen

- To lock the touchscreen, select the Unlock quick key  and swipe the slider upward as instructed
- To unlock the touchscreen, select the Lock quick key  and swipe the slider upward as instructed

Please note: The touchscreen will lock automatically after 60 seconds

These guidelines do NOT replace the instructions for use and all users should refer to the appropriate Operator's Manual for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.

Quick key functions




Alarm Reset

Acknowledges ongoing alarms and resets the alarm system




Alarm Pause

Pauses the current alarms

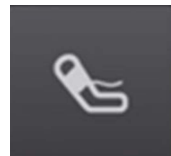

NIBP Start / Stop

Starts an NIBP measurement or stops the current NIBP measurement

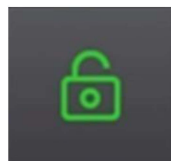

Unlock / Lock

Selects and operates as instructed to Unlock and Lock the touchscreen



Main Menu

Enters the main menu

Touch screen gesture control



Scrolling up and down (eg. review)



Scrolling left and right (eg. Menu pages)



Screen switch



Discharge patient

These guidelines do NOT replace the instructions for use and all users should refer to the appropriate Operator's Manual for detailed instructions.
The Illustrations may not necessarily reflect the setup or data displayed on your device.