


# T1 Monitor Quick Guide

mindray

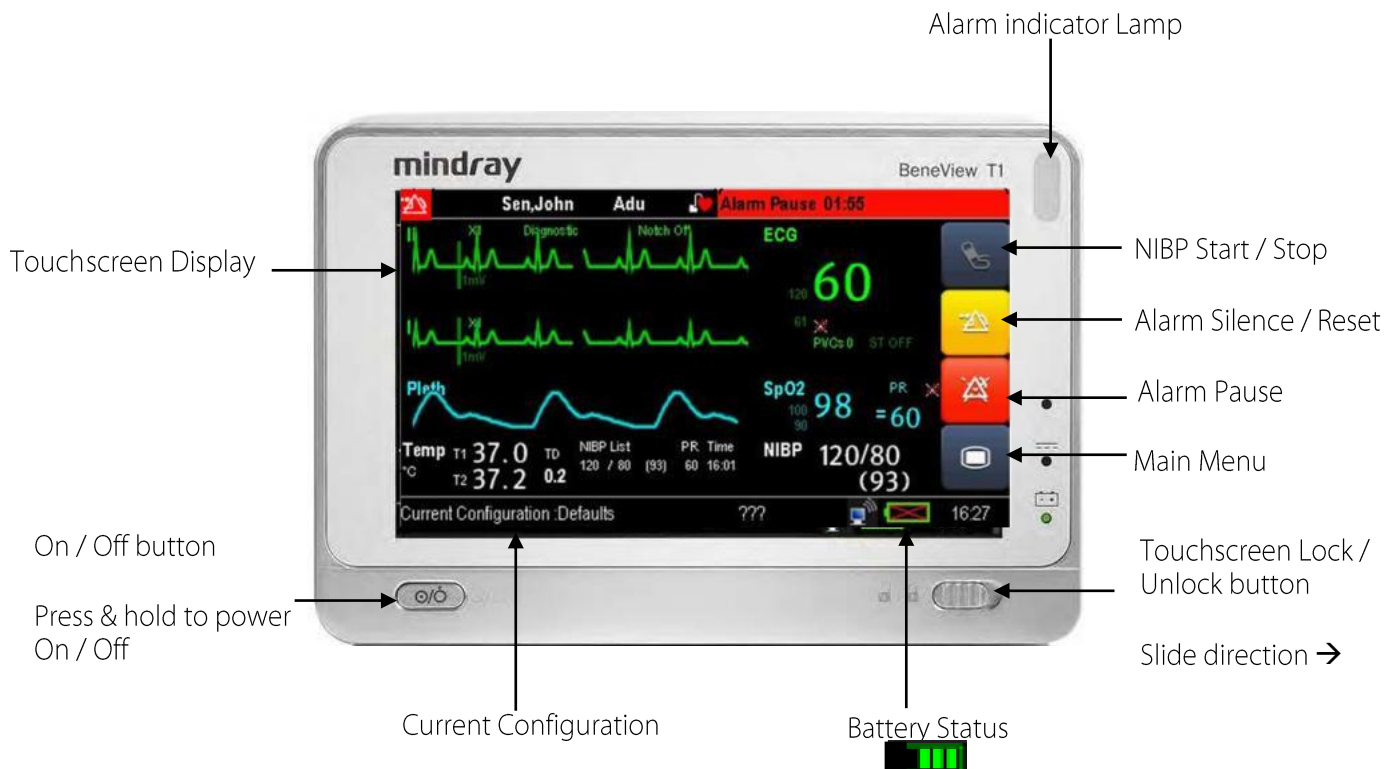

## Removing the T1 from the host monitor

- Press and hold the latch at the bottom of the T1
- Slide the T1 from the host monitor

## Inserting the T1 into the host monitor

- Slide the T1 into the docking area located to the side of the host monitor
- An audible click indicates that the T1 is docked

## To Lock or unlock the touch screen

- Slide the switch  →
- A red padlock symbol appears in the Main Menu key 

**Please note:** The touchscreen will lock automatically after 2 minutes

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.  
The illustrations may not necessarily reflect the setup or data displayed on your device.

## Key Functions



Starts and stops NIBP measurements



Silences the current alarming event and cancels technical alarms


The Alarm Reset symbol appears on the screen 



Mutes all alarms for 2 minutes



Key illuminates red when selected

The Alarm Pause symbol appears on the screen 



Enter and Exit the Main Menu

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.  
The Illustrations may not necessarily reflect the setup or data displayed on your device.