

# Passport 8/12 Bedside Monitor Default Settings

Hospital, City, ST \_\_\_\_\_ Department \_\_\_\_\_

Clinical/Sales/Service Representatives \_\_\_\_\_

p

Date \_\_\_\_\_ Software Version \_\_\_\_\_ Signature, Title \_\_\_\_\_

Maintenance				
System time	Date format:	Time format: 12 <input type="checkbox"/> 24 <input type="checkbox"/>		
Manage Config (See Configurations)				
Department: General	Select default: Load latest <input type="checkbox"/> Load specific <input type="checkbox"/>	User config: User config: User config:		
User Maintenance				
Facility	Room	Change Pt Loc	Monitor name	Visit #
Dept	Bed	On <input type="checkbox"/> Off <input type="checkbox"/>	Language: English	On <input type="checkbox"/> Off <input type="checkbox"/>
Units	Ht In <input type="checkbox"/> Cm <input type="checkbox"/>	CVP mmHg <input type="checkbox"/> kPA <input type="checkbox"/> cm/H2O <input type="checkbox"/>		
	Wt Lbs <input type="checkbox"/> Kg <input type="checkbox"/>	CO2 mmHg <input type="checkbox"/> kPA <input type="checkbox"/> % <input type="checkbox"/>		
	ST mm <input type="checkbox"/> mV <input type="checkbox"/>	O2 mmHg <input type="checkbox"/> kPA <input type="checkbox"/> % <input type="checkbox"/>		
	Pressure mmHg <input type="checkbox"/> kPA <input type="checkbox"/>	Temp F <input type="checkbox"/> C <input type="checkbox"/>		
Network Setup	Select CMS Off <input type="checkbox"/> On <input type="checkbox"/> Central station setup: Network Encryption On <input type="checkbox"/> Off <input type="checkbox"/>			
	Min Alarm Vol	ECG Lead off level Low <input type="checkbox"/> Med <input type="checkbox"/> High <input type="checkbox"/>		
	Alarm Pause	SpO2 sensor off level Low <input type="checkbox"/> Med <input type="checkbox"/> High <input type="checkbox"/>		
	Alarm Sounds ISO <input type="checkbox"/> Mode1 <input type="checkbox"/> Mode2 <input type="checkbox"/>	IBP sensor off level Low <input type="checkbox"/> Med <input type="checkbox"/> High <input type="checkbox"/>		
	High Interval	Lethal Arr off Disable <input type="checkbox"/> Enable <input type="checkbox"/>		
	Med Interval	Extended arrh Disable <input type="checkbox"/> Enable <input type="checkbox"/>		
	Low Interval	Alarm light on Alarm Reset On <input type="checkbox"/> Off <input type="checkbox"/>		
	Reset Other Bed's alarms On <input type="checkbox"/> Off <input type="checkbox"/>	Alarm delay		
	Alarm reset by other bed On <input type="checkbox"/> Off <input type="checkbox"/>	ST alarm delay		
	Other bed disconnect On <input type="checkbox"/> Off <input type="checkbox"/>	Latching Alarms: Lethal Visual <input type="checkbox"/> Audible <input type="checkbox"/>		
	Reminder Interval 1 min <input type="checkbox"/> 2min <input type="checkbox"/> 3min <input type="checkbox"/>	High Visual <input type="checkbox"/> Audible <input type="checkbox"/>		
	Reminder tone On <input type="checkbox"/> Off <input type="checkbox"/> Re-alarm <input type="checkbox"/>	Medium Visual <input type="checkbox"/> Audible <input type="checkbox"/>		
		Low Visual <input type="checkbox"/> Audible <input type="checkbox"/>		
	Vtach Latch lock: Disable <input type="checkbox"/> Enable <input type="checkbox"/>			
Others	Wave line Thin <input type="checkbox"/> Med <input type="checkbox"/> Thick <input type="checkbox"/>	ECG standard: AHA		
	Nurse call set up:	Notch Freq 60 <input type="checkbox"/> 50 <input type="checkbox"/>		
	Signal type: Continuous <input type="checkbox"/> Pulse <input type="checkbox"/>	Data transfer method: Off <input type="checkbox"/> USB <input type="checkbox"/>		
	Contact type: Open <input type="checkbox"/> Closed <input type="checkbox"/>	Transfer data length: 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 4hr <input type="checkbox"/> 8hr <input type="checkbox"/>		
	Alarm level: High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>	SpO2 tone: Mode1 <input type="checkbox"/> Mode 2 <input type="checkbox"/>		
	Alarm cat: Tech <input type="checkbox"/> Phys <input type="checkbox"/>	Clear CMS IP at startup: On <input type="checkbox"/> Off <input type="checkbox"/>		
	DIAP Baud: 9600 <input type="checkbox"/> 19200 <input type="checkbox"/>	Config-influenced settings: Param switch On <input type="checkbox"/> Off <input type="checkbox"/>		
Factory Maintenance				
Central Hypervisor <input type="checkbox"/> Panorama <input type="checkbox"/>	Network setup:			
HR alarm delay On <input type="checkbox"/> Off <input type="checkbox"/>	Presence mode: Broadcast <input type="checkbox"/> Multicast <input type="checkbox"/>			
	HL7 Compatibility On <input type="checkbox"/> Off <input type="checkbox"/>			

## Passport 8/12 Bedside Monitor Default Settings

Alarm Setup—See Configurations																	
Review																	
Graphic Trends	Group: Standard <input type="checkbox"/> Other <input type="checkbox"/> _____ Custom <input type="checkbox"/> List:																
	Zoom: 4min <input type="checkbox"/> 20min <input type="checkbox"/> 40min <input type="checkbox"/> 2hr <input type="checkbox"/> 4hr <input type="checkbox"/> 8hr <input type="checkbox"/> 16hr <input type="checkbox"/> 32hr <input type="checkbox"/>																
	#Waves:																
Tabular Trends	Group: Standard <input type="checkbox"/> Other <input type="checkbox"/> _____ Custom <input type="checkbox"/> List:																
	Interval: 5s <input type="checkbox"/> 30s <input type="checkbox"/> 1min <input type="checkbox"/> 5min <input type="checkbox"/> 10min <input type="checkbox"/> 15min <input type="checkbox"/> 30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> NIBP <input type="checkbox"/>																
Disclosure	Saved waves: _____ Beat Annotation On <input type="checkbox"/> Off <input type="checkbox"/>																
Screen Setup																	
Help On <input type="checkbox"/> Off <input type="checkbox"/>	Key Volume (1-10) _____																
Brightness(1-10) _____																	
Screen Layout (Normal/Big Numeric Setup--see Configurations)																	
Minitrends Setup	<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td rowspan="5" style="width: 20%; text-align: center; vertical-align: middle;">PP8/12</td> <td style="width: 30%;">Parameter 1</td> <td style="width: 50%;"></td> </tr> <tr> <td>Parameter 2</td> <td></td> </tr> <tr> <td>Parameter 3</td> <td></td> </tr> <tr> <td>Parameter 4</td> <td></td> </tr> <tr> <td>Parameter 5</td> <td></td> </tr> <tr> <td rowspan="2" style="text-align: center; vertical-align: middle;">PP12</td> <td>Parameter 6</td> <td></td> </tr> <tr> <td>Parameter 7</td> <td></td> </tr> </table>	PP8/12	Parameter 1		Parameter 2		Parameter 3		Parameter 4		Parameter 5		PP12	Parameter 6		Parameter 7	
PP8/12	Parameter 1																
	Parameter 2																
	Parameter 3																
	Parameter 4																
	Parameter 5																
PP12	Parameter 6																
	Parameter 7																
<b>Factory Defaults</b> <input type="checkbox"/>	Length: 30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 4hr <input type="checkbox"/> 8hr <input type="checkbox"/>																
OxyCRG Setup	Time: 1 min <input type="checkbox"/> 2 min <input type="checkbox"/> 4 min <input type="checkbox"/> 8 min <input type="checkbox"/>																
<b>Factory Defaults</b> <input type="checkbox"/>	Trend1: HR <input type="checkbox"/> HR+RR <input type="checkbox"/> Trend2: SpO2 <input type="checkbox"/> SpO2b <input type="checkbox"/> SpO2 +SpO2b <input type="checkbox"/> Compressed: Resp <input type="checkbox"/> CO2 <input type="checkbox"/> Event: 1min+3min <input type="checkbox"/> 2min+2min <input type="checkbox"/> 3min+1min <input type="checkbox"/> Scaling: HR: Upper _____ Lower _____ RR: Upper _____ Lower _____ SPO2: Upper _____ Lower _____ CO2: Upper _____ Lower _____																
View Others Setup	Alert tone: Repeat <input type="checkbox"/> Once <input type="checkbox"/> Off <input type="checkbox"/>																

## Passport 8/12 Bedside Monitor Default Settings

Screen Setup cont'd					
Measurement Color Setup					
	Parameter	Color			Factory default
		Adu	Ped	Neo	
White	ECG				<input type="checkbox"/>
Gray 1-3	SPO2				<input type="checkbox"/>
Red 1-3	NIBP				<input type="checkbox"/>
Orange 1-3	Temp				<input type="checkbox"/>
Yellow 1-3	P1				<input type="checkbox"/>
Green 1-3	P2				<input type="checkbox"/>
Cyan 1-3	P3				<input type="checkbox"/>
Blue 1-3	P4				<input type="checkbox"/>
Purple 1-3	Resp				<input type="checkbox"/>
	CO2				<input type="checkbox"/>
	AA				<input type="checkbox"/>
	O2				<input type="checkbox"/>
	N2O				<input type="checkbox"/>
	CO2 (AG)				<input type="checkbox"/>
	CO				<input type="checkbox"/>
					<input type="checkbox"/>
					<input type="checkbox"/>
Night Mode					
Brightness		Volume: Alm      QRS      Key		NIBP End tone: On <input type="checkbox"/> Off <input type="checkbox"/> Stop NIBP <input type="checkbox"/>	
Record Setup					
Waveforms	Wave 1				
	Wave 2				
	Wave 3				
	IBP Overlap				
Length	8 sec <input type="checkbox"/> Continuous <input type="checkbox"/>				
Interval	Off <input type="checkbox"/> 10 min <input type="checkbox"/> 15 min <input type="checkbox"/> 20 min <input type="checkbox"/> 30 min <input type="checkbox"/> 40 min <input type="checkbox"/> 50 min <input type="checkbox"/> 1 hr <input type="checkbox"/> 2 hr <input type="checkbox"/> 3 hr <input type="checkbox"/> 4 hr <input type="checkbox"/>				
Paper Speed	25 mm/s <input type="checkbox"/> 50 mm/s <input type="checkbox"/>				

## Passport 8/12 Bedside Monitor Default Settings

Print Setup			
ECG Reports  <b>Factory Defaults</b> <input type="checkbox"/>		Normal	End of Case Yes <input type="checkbox"/> No <input type="checkbox"/>
	Amplitude	5mm <input type="checkbox"/> 10mm <input type="checkbox"/> 20mm <input type="checkbox"/>	5mm <input type="checkbox"/> 10mm <input type="checkbox"/> 20mm <input type="checkbox"/>
	Auto Interval	On <input type="checkbox"/> Off <input type="checkbox"/>	On <input type="checkbox"/> Off <input type="checkbox"/>
	Sweep	25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>	25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	12 Lead	12x1 <input type="checkbox"/> 6x2 <input type="checkbox"/> 3x4+1 <input type="checkbox"/>	12x1 <input type="checkbox"/> 6x2 <input type="checkbox"/> 3x4+1 <input type="checkbox"/>
Tabular Trends Reports			
<b>Factory Defaults</b> <input type="checkbox"/>		Normal	End of Case Yes <input type="checkbox"/> No <input type="checkbox"/>
	Back	30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> 6hr <input type="checkbox"/> 12hr <input type="checkbox"/> 24hr <input type="checkbox"/> 48h <input type="checkbox"/> 72hr <input type="checkbox"/> 96hr <input type="checkbox"/> Auto <input type="checkbox"/> All <input type="checkbox"/>	30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> 6hr <input type="checkbox"/> 12hr <input type="checkbox"/> 24hr <input type="checkbox"/> 48h <input type="checkbox"/> 72hr <input type="checkbox"/> 96hr <input type="checkbox"/> Auto <input type="checkbox"/> All <input type="checkbox"/>
	Spacing	5s <input type="checkbox"/> 30s <input type="checkbox"/> 1min <input type="checkbox"/> 5min <input type="checkbox"/> 10min <input type="checkbox"/> 15min <input type="checkbox"/> 30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> NIBP <input type="checkbox"/> Auto <input type="checkbox"/>	5s <input type="checkbox"/> 30s <input type="checkbox"/> 1min <input type="checkbox"/> 5min <input type="checkbox"/> 10min <input type="checkbox"/> 15min <input type="checkbox"/> 30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> NIBP <input type="checkbox"/> Auto <input type="checkbox"/>
	Report layout	Parameter oriented <input type="checkbox"/> Time oriented <input type="checkbox"/>	Parameter oriented <input type="checkbox"/> Time oriented <input type="checkbox"/>
	Select parameters	Currently displayed <input type="checkbox"/> Standard group <input type="checkbox"/> Custom <input type="checkbox"/> List:	Currently displayed <input type="checkbox"/> Standard group <input type="checkbox"/> Custom <input type="checkbox"/> List:
Graphic Trends Reports			
<b>Factory Defaults</b> <input type="checkbox"/>		Normal	End of Case Yes <input type="checkbox"/> No <input type="checkbox"/>
	Back	30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> 6hr <input type="checkbox"/> 12hr <input type="checkbox"/> 24hr <input type="checkbox"/> 48h <input type="checkbox"/> 72hr <input type="checkbox"/> 96hr <input type="checkbox"/> Auto <input type="checkbox"/> All <input type="checkbox"/>	30min <input type="checkbox"/> 1hr <input type="checkbox"/> 2hr <input type="checkbox"/> 3hr <input type="checkbox"/> 6hr <input type="checkbox"/> 12hr <input type="checkbox"/> 24hr <input type="checkbox"/> 48h <input type="checkbox"/> 72hr <input type="checkbox"/> 96hr <input type="checkbox"/> Auto <input type="checkbox"/> All <input type="checkbox"/>
	Paginal Time	4min <input type="checkbox"/> 20min <input type="checkbox"/> 40min <input type="checkbox"/> 2hr <input type="checkbox"/> 4hr <input type="checkbox"/> 8hr <input type="checkbox"/> 16hr <input type="checkbox"/> 32hr <input type="checkbox"/> Auto <input type="checkbox"/>	4min <input type="checkbox"/> 20min <input type="checkbox"/> 40min <input type="checkbox"/> 2hr <input type="checkbox"/> 4hr <input type="checkbox"/> 8hr <input type="checkbox"/> 16hr <input type="checkbox"/> 32hr <input type="checkbox"/> Auto <input type="checkbox"/>
	Select parameters	Currently displayed <input type="checkbox"/> Standard group <input type="checkbox"/> Custom <input type="checkbox"/> List:	Currently displayed <input type="checkbox"/> Standard group <input type="checkbox"/> Custom <input type="checkbox"/> List:
Realtime Reports			
<b>Factory Defaults</b> <input type="checkbox"/>		Normal	End of Case Yes <input type="checkbox"/> No <input type="checkbox"/>
	Sweep	12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/> Auto <input type="checkbox"/>	12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/> Auto <input type="checkbox"/>
Select Wave	Current <input type="checkbox"/> Select waves <input type="checkbox"/> List:	Current <input type="checkbox"/> Select waves <input type="checkbox"/> List:	
Print Setup			
	Printer		
	Paper Size	A4 <input type="checkbox"/> Letter <input type="checkbox"/>	
	Print Both Sides	On <input type="checkbox"/> Off <input type="checkbox"/>	
Mark Event			
<b>Factory Defaults</b> <input type="checkbox"/>	Waveforms	Wave 1	ECG1
		Wave 2	
		Wave 3	

## Passport 8/12 Bedside Monitor Default Settings

Parameters		
ECG Setup	Alm Source	
	ECG1	
	ECG2	
	ECG1Gain	
	ECG2Gain	
	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	Diagnostic <input type="checkbox"/> Monitor <input type="checkbox"/> Surgery <input type="checkbox"/> ST <input type="checkbox"/>
	Arrhythmia analysis—See alarm setup	
	ST analysis	On <input type="checkbox"/> Off <input type="checkbox"/>
	QT analysis	On <input type="checkbox"/> Off <input type="checkbox"/> Formula: _____ Lead All <input type="checkbox"/> Single: _____
	Others	QRS Vol (0-10) _____
		Lead Set <b>Adu:</b> Auto <input type="checkbox"/> 3 <input type="checkbox"/> 5 <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>
		ECG Display      Normal <input type="checkbox"/> Full screen <input type="checkbox"/> Half screen <input type="checkbox"/>
		Notch Filter      Off <input type="checkbox"/> Weak <input type="checkbox"/> Strong <input type="checkbox"/>
		Paced      No <input type="checkbox"/> Yes <input type="checkbox"/> Pacer Reject On <input type="checkbox"/> Off <input type="checkbox"/> Pace Rate(40-100)
	Smart Lead Off      On <input type="checkbox"/> Off <input type="checkbox"/>	
	QRS Threshold _____	
Resp Setup	Apnea Delay	<b>Adu:</b> 10 <input type="checkbox"/> 15 <input type="checkbox"/> 20 <input type="checkbox"/> 25 <input type="checkbox"/> 30 <input type="checkbox"/> 35 <input type="checkbox"/> 40 <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>
	Resp Lead	<b>Adu:</b> I <input type="checkbox"/> II <input type="checkbox"/> Auto <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>
	Gain	
	Sweep	3mm/s <input type="checkbox"/> 6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/>
	Detection mode	Auto <input type="checkbox"/> Manual <input type="checkbox"/>
	RR Source	Auto <input type="checkbox"/> ECG <input type="checkbox"/> CO2 <input type="checkbox"/>
SpO2 Setup	Massimo Avg time: 2-4s <input type="checkbox"/> 4-6s <input type="checkbox"/> 8s <input type="checkbox"/> 10s <input type="checkbox"/> 12s <input type="checkbox"/> 14s <input type="checkbox"/> 16s <input type="checkbox"/> Sensitivity Mode: High <input type="checkbox"/> Normal <input type="checkbox"/>	
		Nelcor Sat Sec: 0 <input type="checkbox"/> 10 <input type="checkbox"/> 25 <input type="checkbox"/> 50 <input type="checkbox"/> 100 <input type="checkbox"/>
	NIBP Simul	On <input type="checkbox"/> Off <input type="checkbox"/>
	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	PR source	SpO2 <input type="checkbox"/> IBP <input type="checkbox"/> Art <input type="checkbox"/> Auto <input type="checkbox"/>
	Alm source	HR <input type="checkbox"/> PR <input type="checkbox"/> Auto <input type="checkbox"/>
	Beat vol (0-10)	
NIBP Setup	Initial Pressure	<b>Adu:</b> <b>Ped:</b> <b>Neo:</b>
	Interval	<b>Adu:</b> Manual <input type="checkbox"/> 1m <input type="checkbox"/> 2m <input type="checkbox"/> 2.5m <input type="checkbox"/> 3m <input type="checkbox"/> 5m <input type="checkbox"/> 10m <input type="checkbox"/> 15m <input type="checkbox"/> 20m <input type="checkbox"/> 30m <input type="checkbox"/> 1h <input type="checkbox"/> 1.5h <input type="checkbox"/> 2h <input type="checkbox"/> 3h <input type="checkbox"/> 4h <input type="checkbox"/> 8h <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>
	End Tone	On <input type="checkbox"/> Off <input type="checkbox"/>
	Clock	On <input type="checkbox"/> Off <input type="checkbox"/>
	Venipuncture	<b>Adu:</b> <b>Ped:</b> <b>Neo:</b>
P1 Setup	Label	
	Scale	Auto <input type="checkbox"/> Manual <input type="checkbox"/> Upper _____ Lower _____
	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>
	Sensitivity	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>
	PPV	On <input type="checkbox"/> Off <input type="checkbox"/> Source: _____
P2 Setup	Label	
	Scale	Auto <input type="checkbox"/> Manual <input type="checkbox"/> Upper _____ Lower _____
	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>
	Sensitivity	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>
	PPV	On <input type="checkbox"/> Off <input type="checkbox"/> Source: _____
<b>Factory Defaults</b> <input type="checkbox"/>	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>
	Sensitivity	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>
	PPV	On <input type="checkbox"/> Off <input type="checkbox"/> Source: _____
	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>

## Passport 8/12 Bedside Monitor Default Settings

Parameters cont'd			
<b>*P3 Setup</b>	Label		
	Scale	Auto <input type="checkbox"/> Manual <input type="checkbox"/> Upper _____ Lower _____	
	<b>Factory Defaults</b> <input type="checkbox"/>	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>	
	Sensitivity	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>	
	PPV	On <input type="checkbox"/> Off <input type="checkbox"/> Source:	
<b>*P4 Setup</b>	Label	Measure: All <input type="checkbox"/> Mean <input type="checkbox"/>	
	Scale	Auto <input type="checkbox"/> Manual <input type="checkbox"/> Upper _____ Lower _____	
	<b>Factory Defaults</b> <input type="checkbox"/>	Sweep	6.25mm/s <input type="checkbox"/> 12.5mm/s <input type="checkbox"/> 25mm/s <input type="checkbox"/> 50mm/s <input type="checkbox"/>
	Filter	8Hz <input type="checkbox"/> 12.5Hz <input type="checkbox"/> 20Hz <input type="checkbox"/> 40Hz <input type="checkbox"/>	
	Sensitivity	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>	
	PPV	On <input type="checkbox"/> Off <input type="checkbox"/> Source:	
<b>*CO2 Setup</b>	Apnea Delay	<b>Adu:</b> 10 <input type="checkbox"/> 15 <input type="checkbox"/> 20 <input type="checkbox"/> 25 <input type="checkbox"/> 30 <input type="checkbox"/> 35 <input type="checkbox"/> 40 <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>	
	BTPS Comp	On <input type="checkbox"/> Off <input type="checkbox"/>	
	Sidestream	O2 Comp	
		N2O Comp	
		Des Comp	
		Flow Rate	70 <input type="checkbox"/> 100 <input type="checkbox"/> 120 <input type="checkbox"/> 150 <input type="checkbox"/> (ml/min)
	Oridion	Max Hold	Single <input type="checkbox"/> 10 <input type="checkbox"/> 20 <input type="checkbox"/> 30 <input type="checkbox"/>
	Op mode		Standby <input type="checkbox"/> Measure <input type="checkbox"/>
	Wave type		Draw <input type="checkbox"/> Fill <input type="checkbox"/>
	Scale mmHg kPA %		
	Sweep		6.25 <input type="checkbox"/> 12.5 <input type="checkbox"/> 25 <input type="checkbox"/> (mm/s)
	RR source		Auto ECG
	Auto Standby		Off <input type="checkbox"/> 15m <input type="checkbox"/> 30m <input type="checkbox"/> 60m <input type="checkbox"/>
<b>*AG Setup</b>	Apnea Delay	<b>Adu:</b> 10 <input type="checkbox"/> 15 <input type="checkbox"/> 20 <input type="checkbox"/> 25 <input type="checkbox"/> 30 <input type="checkbox"/> 35 <input type="checkbox"/> 40 <input type="checkbox"/> <b>Ped:</b> <b>Neo:</b>	
	Flow rate	High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>	
	Op mode	Standby <input type="checkbox"/> Measure <input type="checkbox"/>	
	CO2 Wave Type	Draw <input type="checkbox"/> Fill <input type="checkbox"/>	
	CO2 Scale mmHg kPA %		
	AA Scale		
	N2O Scale %		
	O2 Scale mmHg kPA %		
	Sweep		6.25 <input type="checkbox"/> 12.5 <input type="checkbox"/> 25 <input type="checkbox"/> (mm/s)
<b>Factory Defaults</b> <input type="checkbox"/>			



## Passport 8/12 Bedside Monitor Default Settings

Configuration Name <span style="float: right;">(Ped)</span>											
Alarm Setup: Alarm report attached <input type="checkbox"/>											
Parameter	*On/Off	High	Low	Level	Record	Other	*On/Off	High	Low	Level	Record
HR/PR						*ST I					
Ext Tachy						*ST II					
Ext Brady						*ST III					
QTc						*ST avR					
ΔQTc						*ST avL					
RR						*ST avF					
SpO2						*ST V					
Desat						*EtCO2					
NIBP-S						*FiCO2					
NIBP-D						*AwRR					
NIBP-M						*AA					
T1						*P-S					
T2						*P-D					
TD						*P-M					
P1-S						*P-S					
P1-D						*P-D					
P1-M						*P-M					
P2-S											
P2-D											
P2-M											
*Arrhythmia	On/Off	Level	Rec	*Arrhythmia	On/Off	Level	Rec	*Arrhythmia Threshold			
Asystole				Bigeminy				PVC's High			
VT/VFib				Trigeminy				Extreme Tachy			
VTach				Tachy				Tachy (HR High)			
*VBrady				Brady				Brady (HR low)			
*Extreme Tachy				*PNP				Extreme Brady			
*ExtremeBrady				*PNC				Asys delay			
PVC's/min				Missed Beat				Multi PVC			
R on T				*Nonsust VT				VTach rate			
*Run PVCs				Vent Rhythm				VTach PVCs			
Couplet				*Pause				Pause time			
Multi PVC				Irr Rhythm				VBrady rate			
*PVC				*AFib				VBrady PVCs			
								<b>Others</b>			
								Alm Vol (1-10)			
								High: Alm Vol + 0 <input type="checkbox"/> 1 <input type="checkbox"/> 2 <input type="checkbox"/>			
								Reminder Vol High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>			
								Record length 8s <input type="checkbox"/> 16s <input type="checkbox"/> 32s <input type="checkbox"/>			
								Apnea delay			
								Tech delay Off <input type="checkbox"/> 5s <input type="checkbox"/> 10s <input type="checkbox"/> 15s <input type="checkbox"/>			
								Vtac Latching On <input type="checkbox"/> Off <input type="checkbox"/>			



## Passport 8/12 Bedside Monitor Default Settings

Configuration Name											
Alarm Setup: Alarm report attached <input type="checkbox"/>											
Parameter	*On/Off	High	Low	Level	Record	Other	*On/Off	High	Low	Level	Record
HR/PR						*ST I					
Ext Tachy						*ST II					
Ext Brady						*ST III					
QTc						*ST avR					
ΔQTc						*ST avL					
RR						*ST avF					
SpO2						*ST V					
Desat						*EtCO2					
NIBP-S						*FiCO2					
NIBP-D						*AwRR					
NIBP-M						*AA					
T1						*P-S					
T2						*P-D					
TD						*P-M					
P1-S						*P-S					
P1-D						*P-D					
P1-M						*P-M					
P2-S											
P2-D											
P2-M											
*Arrhythmia	On/Off	Level	Rec	*Arrhythmia	On/Off	Level	Rec	*Arrhythmia Threshold			
Asystole				Bigeminy				PVC's High			
VT/VFib				Trigeminy				Extreme Tachy			
VTach				Tachy				Tachy (HR High)			
*VBrady				Brady				Brady (HR low)			
*Extreme Tachy				*PNP				Extreme Brady			
*ExtremeBrady				*PNC				Asys delay			
PVC's/min				Missed Beat				Multi PVC			
R on T				*Nonsust VT				VTach rate			
*Run PVCs				Vent Rhythm				VTach PVCs			
Couplet				*Pause				Pause time			
Multi PVC				Irr Rhythm				VBrady rate			
*PVC				*AFib				VBrady PVCs			
								<b>Others</b>			
								Alm Vol (1-10)			
								High: Alm Vol + 0 <input type="checkbox"/> 1 <input type="checkbox"/> 2 <input type="checkbox"/>			
								Reminder Vol High <input type="checkbox"/> Med <input type="checkbox"/> Low <input type="checkbox"/>			
								Record length 8s <input type="checkbox"/> 16s <input type="checkbox"/> 32s <input type="checkbox"/>			
								Apnea delay			
								Tech delay Off <input type="checkbox"/> 5s <input type="checkbox"/> 10s <input type="checkbox"/> 15s <input type="checkbox"/>			
								Vtac Latching On <input type="checkbox"/> Off <input type="checkbox"/>			

## Passport 8/12 Bedside Monitor Default Settings

Screen Setup																																				
Choose Screen	Normal <input type="checkbox"/>	OxyCRG <input type="checkbox"/>	ECG 7 Ld Half <input type="checkbox"/>	PAWP <input type="checkbox"/>																																
	Big Numerics <input type="checkbox"/>	View Others <input type="checkbox"/>	ECG 7 Ld Full <input type="checkbox"/>	CO <input type="checkbox"/>																																
	Minitrends <input type="checkbox"/>																																			
Normal Screen Setup	<table border="1" style="margin: auto; border-collapse: collapse;"> <tr> <td style="width: 20%; text-align: center;">Wave 1</td> <td colspan="3" style="text-align: center;">ECG 1</td> </tr> <tr style="background-color: #f2f2f2;"> <td style="text-align: center;">Wave 2</td> <td colspan="3"></td> </tr> <tr> <td style="text-align: center;">Wave 3</td> <td colspan="3"></td> </tr> <tr style="background-color: #f2f2f2;"> <td style="text-align: center;">Wave 4</td> <td colspan="3"></td> </tr> <tr> <td style="text-align: center;">Wave 5</td> <td colspan="3"></td> </tr> <tr style="background-color: #f2f2f2;"> <td style="text-align: center;">Wave 6</td> <td colspan="3"></td> </tr> <tr> <td style="text-align: center;">Wave 7 Parameter</td> <td></td> <td></td> <td></td> </tr> <tr style="background-color: #f2f2f2;"> <td style="text-align: center;">Wave 8 Parameter</td> <td></td> <td></td> <td></td> </tr> </table>				Wave 1	ECG 1			Wave 2				Wave 3				Wave 4				Wave 5				Wave 6				Wave 7 Parameter				Wave 8 Parameter			
Wave 1	ECG 1																																			
Wave 2																																				
Wave 3																																				
Wave 4																																				
Wave 5																																				
Wave 6																																				
Wave 7 Parameter																																				
Wave 8 Parameter																																				
Big Numerics	Param 1		Param 3																																	
	Param 2		Param 4																																	
Quick Keys	First Page		Second Page																																	
Parameters Switch	<i>All On</i> <input type="checkbox"/> <i>On except (List):</i>																																			
<b>Parameters-- see above</b>																																				